
Akku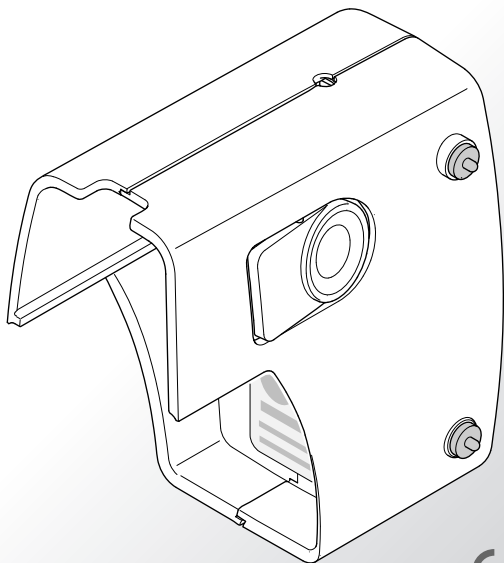

de	Gebrauchsanweisung.....	3
en	Instructions for use	17

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Instructions for use

Information as of: 2014-07. Subject to technical changes.

IMPORTANT INFORMATION	18
General.....	18
Structure of safety instructions	18
Using the rechargeable battery	18
PRODUCT DESCRIPTION	19
Components	19
Intended use.....	19
Explanation of the LEDs on the rechargeable battery.....	19
Product combinations	20
Operating life	20
PREPARATION FOR USE	21
Charging the rechargeable battery	21
Connecting the rechargeable battery to the compressor .	22
Disconnecting the rechargeable battery from the compressor.....	23
MAINTENANCE	23
Cleaning the rechargeable battery	23
Storage.....	24
TROUBLESHOOTING	25
TECHNICAL DATA	26
General.....	26
Ambient conditions for operation	26
MISCELLANEOUS	27
Transportation and storage	27
Terms and conditions of warranty	27
Manufacturer information	29
Contact	29
Explanation of symbols	29

1 IMPORTANT INFORMATION

1.1 General

Please read these instructions for use carefully and completely. Do not discard, so you can consult them at a later date. If you fail to comply with the instructions for use, injury or damage to the device cannot be ruled out.

1.2 Structure of safety instructions

Safety-critical warnings are categorised according to hazard levels in these instructions for use:

- The signal word **WARNING** is used to indicate hazards which, without precautionary measures, can result in serious injury or even death.
- The signal word **CAUTION** is used to indicate hazards which, without precautionary measures, can result in minor to moderate injury or impair treatment.
- The signal word **NOTICE** is used to indicate general precautionary measures which are to be observed to avoid damaging the product during use.

1.3 Using the rechargeable battery

The rechargeable battery must only be used with type 047 PARI compressors. The designator "Type 047" is inscribed on the identification label of suitable compressors.

To ensure problem-free functioning, do not expose the rechargeable battery to direct sunlight, moisture, temperature extremes (above 60 °C), or strong chemicals

The rechargeable battery must not be short circuited. Therefore, ensure that the contacts on the rechargeable battery do not come into contact with electrically conductive metal (e.g., keys).

There is a **risk of explosion** close to open fire.

The rechargeable battery is not splashproof. Protect it from moisture and liquids.

Use only the PARI power adapter or 12 V car adapter supplied with the compressor to charge the rechargeable battery. Use of other products can damage the rechargeable battery.

The rechargeable battery must not be disassembled. Dispose of it in accordance with the regulations prevailing in the country where you are located.

2 PRODUCT DESCRIPTION

2.1 Components

The rechargeable battery is an accessory for type 047 compressors. It is supplied without a charging adapter.

2.2 Intended use

The rechargeable battery is intended to ensure that a regular voltage supply is delivered to PARI type 047 compressors.

2.3 Explanation of the LEDs on the rechargeable battery

The LEDs on the battery indicate its charge level when charging as follows:

Charging process display (goes out when rechargeable battery is fully charged)

Charge level 75% and higher

Charge level 50% to 74%

Charge level 25% to 49%

The LEDs indicate the battery capacity during operation as follows:

— Battery capacity 75% to 100%

— Battery capacity 50% to 74%

— Battery capacity 25% to 49%

The LED flashes when the battery capacity falls below 25%.

When this happens, the remaining runtime is approx. 10 minutes, i.e. the battery must be recharged soon.

If all four LEDs are flashing, there is a fault in the rechargeable battery [see also: 5 TROUBLESHOOTING].

2.4 Product combinations

The rechargeable battery is suitable for operation with all type 047 compressors manufactured by PARI.

2.5 Operating life

If the ambient conditions for [see also: 7.1 Transportation and storage] are complied with, the rechargeable battery has an expected life cycle of:

- approx. 500 charge cycles
- max. 1 year

3 PREPARATION FOR USE

WARNING

Liquids can conduct electricity, thereby posing a **risk of electric shock**. Therefore, never take hold of the power adapter with damp or wet hands.

3.1 Charging the rechargeable battery

The rechargeable battery can be charged using either the power adapter supplied with a type 047 compressor or with the PARI 12 V car adapter, available as an option.


Check the battery housing for damage every time before using. The battery must not be charged or used if it is damaged.

In the delivered state, the rechargeable battery is charged to a level up to 60%. Charge it fully before using for the first time. The charging time for the rechargeable battery depends on its age and residual capacity, but a completely drained battery can be fully recharged in about 2 hours

When fully charged, the new battery has an operating time of about 40 minutes. This corresponds to approx. 8-10 inhalation therapy sessions with a fill quantity of 2.5 ml.


- Connect the power adapter to the power cord socket of the rechargeable battery and then plug it into a suitable socket.

(principle shown in figure)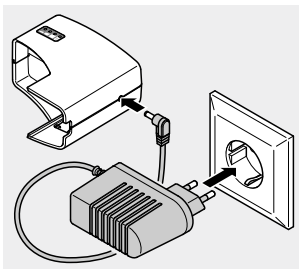


The charging process display lights up, i.e., the rechargeable battery is being charged. As soon as the battery is fully charged, the charging process display goes out [see also: 2.3 Explanation of the LEDs on the rechargeable battery]. When the charging process is complete, disconnect the power adapter from the mains socket first, and then from the battery.

Info: The rechargeable battery can also be charged using the 12 V car adapter: If the compressor is switched off but still connected to a 12 V power supply via the 12 V car adapter, the rechargeable battery is charged automatically.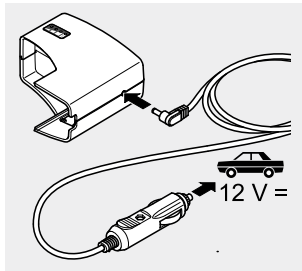


3.2 Connecting the rechargeable battery to the compressor

- Place the rechargeable battery on the compressor. Press the two components together until the engaging hook on the release button engages in the bottom of the battery.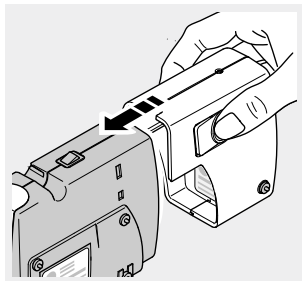

3.3 Disconnecting the rechargeable battery from the compressor

- Disconnect the power adapter or the car adapter from the battery.
- Press the engaging hook inwards and pull the battery away from the compressor.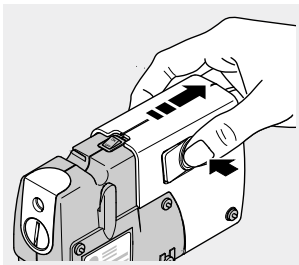

4 MAINTENANCE


4.1 Cleaning the rechargeable battery

WARNING

Liquids can conduct electricity, thereby posing a **risk of electric shock**. Therefore, always switch the compressor off and disconnect the power adapter from the mains socket before starting to clean it.

- Wipe the outer surface of the rechargeable battery with a clean, damp cloth.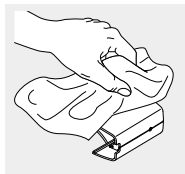

NOTICE

Liquids that get into the device can cause a **fault**. Therefore, do not spray any liquid onto the rechargeable battery.

If liquid does get into the battery, it must not be used under any circumstances. Before using the battery again, you must contact the PARI GmbH Technical Service department or the PARI Service Center [see also: 7.4 Contact].

4.2 Storage

- Disconnect the power adapter or the 12 V car adapter from the battery.
- Store the rechargeable battery away from all heat sources (e.g., continuous direct sunlight).

Always disconnect the rechargeable battery from the mains for storage because electrical devices that remain plugged in present a potential hazard source.

5 TROUBLESHOOTING

In the event of faults that are not listed in this chapter, or if the suggested procedure does not correct the fault, contact the PARI GmbH Service Center.

Fault	Possible cause	Procedure
The rechargeable battery is plugged into the compressor, but the compressor is not working.	The battery is flat.	Connect the power adapter to the rechargeable battery to charge it.
	There is a fault in the compressor or the rechargeable battery.	Reset the rechargeable battery: <ul style="list-style-type: none"> • Switch the compressor off. • Disconnect the battery from the compressor [see also: 3.3 Disconnecting the rechargeable battery from the compressor].
All 4 LEDs are flashing	<ul style="list-style-type: none"> – The battery is overloaded (> 3.5 A). – The battery is overheated. – The battery is flat (deeply discharged). 	<ul style="list-style-type: none"> • Wait at least 10 minutes before operating the rechargeable battery again. • If the problem persists, try charging the battery [see also: 3.1 Charging the rechargeable battery]. • If the problem still cannot be corrected, contact Technical Service [see also: 7.4 Contact].

6 TECHNICAL DATA

6.1 General

Nominal energy	15.6 Wh
Nominal voltage	7.24 V
Nominal capacitance	2,150 mAh
Output extra-low voltage	12 VDC / 1.67 A
Housing dimensions [cm]	(W × H × D) 8.7 × 4.5 × 8.9
Weight	160 g
Runtime	approx. 40 min. when fully charged
Charging time	approx. 2 h. to charge fully
Input	12 V DC / 1.67 A
Output	12 V DC / 1.67 A

6.2 Ambient conditions for operation

Ambient temperature	+10 °C to +30 °C
Relative humidity	30% to 75% (non-condensing)
Atmospheric pressure	700 hPa to 1,060 hPa

7 MISCELLANEOUS

7.1 Transportation and storage

Minimum ambient temperature (without monitoring of relative humidity)	-20 °C
Maximum ambient temperature (with relative humidity of up to 93%, non-condensing)	+45 °C
Humidity	0% – 93%
Atmospheric pressure	500 hPa to 1,060 hPa

7.2 Terms and conditions of warranty

If your rechargeable battery is used in accordance with the instructions, we guarantee to you that it will be free from defects in material and workmanship caused by the manufacturing process for the warranty period indicated on the warranty certificate beginning on the date of initial purchase. Claims under the warranty shall be subject to a limitation period of 12 months. The warranty provided applies in addition to the warranty obligation of your dealer. Your statutory rights with respect to your dealer in the event of defects are not limited by the warranty or any claim under the warranty. The warranty certificate stamped by the dealer serves as your proof of warranty and ownership.

What does the warranty cover?

If, exceptionally, a defect is discovered, at the manufacturer's discretion the rechargeable battery will be repaired or replaced, or the purchase price will be refunded. If it is replaced, the replacement battery will either be the same model or a model that is at least comparably equipped. Replacement or repair of the rechargeable battery shall not serve as the basis for a new warranty. All replaced parts will become the property of the manufacturer. Further claims are excluded. This applies particularly for any claims for compensatory damages. This dis-

claimer of warranty shall be ineffective in the event of injury to life, limb or health, in cases of wilful malfeasance and gross negligence, product liability and if substantive obligations under the warranty agreement are violated.

The warranty shall be cancelled if

- the rechargeable battery has been operated or used improperly with respect to the descriptions in the instructions for use
- damage is present that is attributable to the effects of water, fire, lightning, etc.
- the damage was caused by transporting the device incorrectly or a falling impact
- the rechargeable battery has been misused or not cared for correctly
- the serial number on the rechargeable battery has been changed, removed, or otherwise rendered illegible
- repairs, adaptations or modifications have been made to the rechargeable battery by unauthorised persons

Moreover, the warranty does not cover wearing parts, that is to say device parts that are exposed to normal wear.

In the event of a complaint, please bring the rechargeable battery to your specialist dealer or send it to us packed in the original box, postage paid, together with the warranty certificate stamped by the dealer.

The "warranty period" begins on the date of purchase.

7.3 Manufacturer information

Manufacturer:

VARTA Microbattery GmbH
Daimlerstr. 1
73479 Ellwangen

Sales & Marketing

PARI GmbH
Moosstr. 3
82319 Starnberg

7.4 Contact

For all product information and in the event of defects or questions about usage, please contact our Service Center:

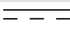
Tel.: +49 (0)8151-279 279 (German-speaking)
+49 (0)8151-279 220 (international)


For repairs and claims under the warranty, please send the device to the following address:

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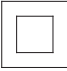7.5 Explanation of symbols

The following symbols can be found on the rechargeable battery and/or the packaging:

	Serial number of the device
	Direct current
	Minimum and maximum ambient temperature

	Minimum and maximum humidity
	Minimum and maximum air pressure
	Protection class II device
	Polarity of contacts on the rechargeable battery
	This product is subject to the provisions of the WEEE (Directive 2002/96/EC on waste electrical and electronic equipment). Accordingly, this product must not be disposed of with domestic waste. The disposal regulations prevailing in the respective member countries must be observed (e.g., disposal by local authorities or dealers).
	Recycling (lithium ions)
	Manufacturer
	ECE approval for operation in motor vehicles.
	This product satisfies the requirements of 2004/108/EC (EMC directive), 2006/66/EC (Battery directive) and 2011/65/EU (RoHS directive).

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Confirmation of purchase: The appliance with the above appliance number was sold in its original packaging.

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CERTIFICATE OF GUARANTEE

*We grant a 6 months guarantee on the accu,
commencing on the date of purchase.*

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©2014 PARI GmbH Spezialisten für effektive Inhalation, 047D3011-B 08/14